

ZUMRO PRODUCT NEWS

INTRODUCING: Zip-On Airlock/Ante-room Shelter

NEW

ZUMRO AIRLOCK SHELTER

With H1N1 and many other possible biological threats, this new Shelter can be “zipped” onto any ZUMRO Shelter or Quad as an add-on Ante-Room offering air-lock entry and egress.

This unit will enable you to use any existing ZUMRO Shelter as a pressure-differential capable Isolation Shelter.



MADE IN AMERICA



FEATURES:

1. 65 Sq. Ft. Shelter fits onto any ZUMRO Shelter that is “Connector Door” equipped.
2. Six adjustable Vents to allow inflow of “make-up” air.
3. Large windows to observe use status.
4. 36” wide Roll-up Doors both ends.
5. Large Air-Lock easily accommodates gurney.
6. Compact Storage of 24” x 24” x 30” and weighs 85 Lbs.
7. Includes Electric 110V (or 220V) Inflator and Air-Bottle Connection Hose.
8. Includes soft sided Carrying Case.
9. Available in all ZUMRO Colors.
10. ZUMRO Part #7650
11. MSRP Price: \$7,388.00

Deploys in 20 Seconds

Operational Sequence



1. Close all doors in Shelter System and proceed to establish required negative pressure with Medical Hepa Scrubber System (s)* in both the Shelter and Airlock (Hepa Unit part #9948).
2. Open exterior roll-up door and enter with patient. (keep door to Shelter closed).
3. Close doors and wait for negative pressure to re-establish in the Airlock Shelter (average time 10 seconds).
4. Open door to the Shelter and enter with patient.
5. Close all doors.

* Note: More than one unit may be required depending on volumetric size of overall Shelter System. Please consult with your ZUMRO authorized representative for details when purchasing system.

ZUMRO, Inc. ~ P.O. Box 696 ~ Hatboro, PA 19040 ~ USA

Phone: 215/957-6502 ~ Toll Free: 800/932-6003 ~ Fax: 215/957-6501 ~ www.zumro.com ~ info@zumro.com